

# OneTen Quik Start Guide

1

Enclosed in your Sharpening System:

- OneTen Sharpening Tool
- Micro-USB charging cord
- Stone Adhesive Tabs
- Quik Start Guide

2

Register your OneTen to access the Online Training

Visit  
[www.dhcubed.com](http://www.dhcubed.com)  
to register

3

Take the UNIVERSAL CURRETTE SHARPENING online training course for your OneTen at:

[www.dhcubed.com](http://www.dhcubed.com)

4

Charge your OneTen fully before first use

Use a phone charging block with the enclosed cable

*The unit is charged when the small red light is no longer showing*

5

Using clean gloves, adhere your OneTen device to your sterilized sharpening stone by using the sticky adhesive or by wrapping the rubber band around the corners of the stone.  
(watch Video Training)

6

Holding the Stone, turn your OneTen on by shaking, until the lights Blink.

(watch Video Training)

7

The OneTen is programmed to light up when you are at the correct sharpening angle of 110-degrees.

(watch Video Training)

8

Your goal is to keep the lights lit up while sharpening

9

The device will turn itself off when it has been idle for one minute.

The lights will blink two times when turning off.

10

Recharge your OneTen once or twice per week.

Do **NOT** charge it daily.

Over charging will reduce the lifespan of the device.

11

Wipe the OneTen with and approved dental disinfectant when changing stoned between each patient use.

12

Visit  
[www.dhcubed.com](http://www.dhcubed.com) for additional information on using your OneTen.